



# User Guide

## Front Panel

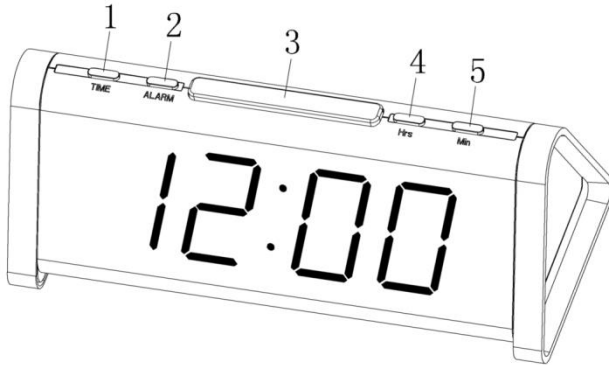

- 1. Time Set Button
- 2. Alarm Set Button
- 3. Snooze Button
- 4. Hour Adjust Button
- 5. Minutes Adjust Button
- 6. Alarm Switch
- 7. Display Dimmer Switch
- 8. Test Switch
- 9. Vibrating Pad Port
- 10. Volume Switch
- 11. USB Port
- 12. Audio Tone Switch
- 13. Power Cable Port
- 14. LED Side Light Switch

## Back Panel

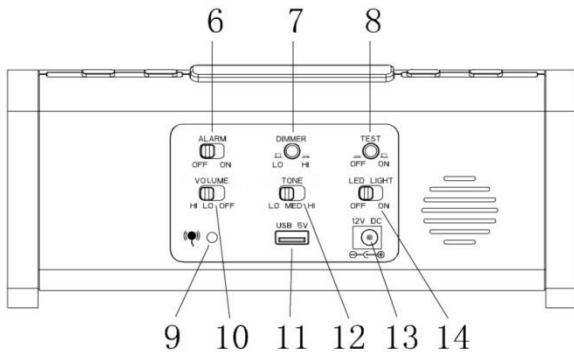

## Instructions:

Insert the Vibrating Pad jack into the Vibrating Pad Port (9) and insert the power supply jack into Power Cable Port (13). Insert the power supply plug into a socket and switch on.

If keeping the clock by your bed we recommend the vibrating pad is placed under a pillow.

### To set time:

Press and hold the Time button (1), followed by the Hour button (4) to change the digit, press several times until you reach the desired hour. Do the same for the minutes by pressing the Minute button (5) together with the Time button (1).

### To set alarm:

Press and hold the Alarm button (2) followed by the Hour (4) and Minutes (5) buttons to the desired time. Set the alarm switch (6) to the on position.

### How to turn off the alarm:

When the alarm goes off, press the Snooze button (3) to snooze the alarm for approx. 9 minutes. You can continue this process 7 times before the alarm mode turns off. To switch the alarm off straight away, or stop snooze mode, press and hold the Snooze button (3) for 3 seconds.

The set alarm will then repeat at the desired time the next day.

## Other functions

### Test the Alarm

You can test the alarm mode by pressing the test button (8) to release. When the button is protruding out the test mode will activate. Push the button back in to turn the test off.

### Charge your Mobile Phone

Insert your mobile phone's USB charger into the USB port (11). Can be used to charge most 5V mobile phones or other electronic devices.

### Adjust the clock display and Alarm Sound

The brightness of the display can be adjusted by pressing the display dimmer switch (7).

You can choose the desired volume of the alarm sound by adjusting the volume switch (10) between HI, LO and OFF. The tone of the alarm sound can also be adjusted using the Audio Tone switch (12) between HI, MED and LO.

The clock's lights can be turned on/off using the LED side light switch (14).

### Power Cut

In the case of a power cut the clock will memorise the time for up to 72 hours so no need to reset the time and it ensures the alarm remains correct while you are sleeping.

### Safety Information

Use only the power adaptor supplied.  
Do not overload power outlets and extension cords. This can result in fire or electric shock.  
Completely insert the power adaptor into the power outlet. Failure to do so may result in electric shock and/or excessive heat causing a fire.  
Regularly remove any dust etc. from the power adaptor by pulling from the power outlet and wiping with a soft dry cloth. Accumulated dust may cause an insulation defect.  
Unplug the product from power outlets if it emits smoke, an abnormal smell or makes an unusual noise.  
Unplug from power outlets and never touch the inside of the product if the case is broken.  
Never touch the plug with wet hands – danger of electric shock exists.  
Always ensure there is a free flow of air over the alarm clock.

### Cleaning

Unplug and clean the alarm clock and vibrating pad with a soft cloth. Do not use polish or other cleaning agents as they may damage the inside or finish of the product.

### Operating Safeguards

Unplug the product from the power outlet before cleaning. Do not use liquid or aerosol cleaners.  
Do not disassemble the product.  
Do not spill liquids (e.g. detergents, cleaners) onto the product or plug, or allow them to become wet. This may cause a fire.

### Volume Warning

This unit has a maximum operating volume of 95dB.

**SS/10700 working temperature** : 0°C to 40°C



### Guarantee

This product is guaranteed for 2 years from the time of purchase. During this time all repairs and replacements (at our discretion) are free of charge. The guarantee excludes general wear and tear and damage to cables.

Important: Your receipt is part of your guarantee and must be retained and produced in the event of a warranty claim.



This product may not be treated as household waste. Instead it should be handed over to the applicable collection point for the recycling of electrical and electronic equipment.

For more information about recycling this product, please contact your local council office or household waste disposal service.

Dispose of batteries in the correct manner via your local waste recycling centre and not in your general waste bin.



## Shake 'n' Wake Extra Loud Alarm Clock

### User Guide

Congratulations on purchasing your Shake 'n' Wake™ Extra Loud Alarm Clock from Safe and Sound Products.

This alarm clock's vibrating pad, flashing lights and extra loud sounder will help to wake even the heaviest sleeper. It also has a jumbo display to help people with poor sight and a USB socket to charge your mobile phone.

### Contents

- Extra Loud Alarm Clock
- Vibrating Pad
- UK Mains Power Supply
- User Guide



Product Code: SS/10700

Made in China

[www.safeandsoundproducts.net](http://www.safeandsoundproducts.net)

---

Safe and Sound Products  
c/o Gordon Ellis & Co.  
Trent Lane, Castle Donington  
Derby, DE74 2AT, UK  
[info@safeandsoundproducts.net](mailto:info@safeandsoundproducts.net)  
01332 856022

