

Contents

- A. Safety instructions**
- B. Introduction and Product features**
- C. Product parts**
- D. Part function**
- E. Set up and user instructions**
- F. Technical specifications**
- G. Troubleshooting**

A. Safety instructions

EchoIR

Aid for Living Infrared System Personal Listener

Please read and follow the safety instructions before using the product

Safety Instructions and Precautions

- * When using this unit, basic safety precautions as below should always be followed to avoid the risk of electrical shock or personal injury.
- * Listening to uncomfortable sound levels for prolonged periods could adversely affect your hearing.
- * If using a microphone or line input, the plug **MUST** be fully inserted into the correct socket and the **volume must always be turned to the lowest level** before turning the unit on. Failure to do so can result in high sound levels that could adversely affect your hearing.
- * Using this product inappropriately could adversely affect your hearing. Please follow the step-by-step instructions carefully!
- * Do not use in an environment that is damp, wet, very hot or very cold.
- * To clean the EchoIR™ and its accessories, use only a soft cloth and an inorganic cleaner **Always remove the power adapter before cleaning the EchoIR.**
- * Only use the Hearing Products Int'l Ltd approved mains adapter. Do not use any other type of mains adapter.
- * Install the unit securely on a stable surface, well ventilated with at least 10 cms free space around the transmitter and install the unit where the power cord will not be subject to damage or cause a tripping hazard.
- * The unit may get warm when being used. Ensure the unit has adequate ventilation.
- * Please refer all servicing to authorized qualified personnel **ONLY!** Do not attempt to open the unit!