


E. Set up and user instructions


1. Place the EchoIR transmitter in well ventilated position on top of or next to your TV ensuring the infra red lens faces into the room, then plug the 12VDC power adapter into an electrical power socket and plug the power lead into the transmitter DC socket.

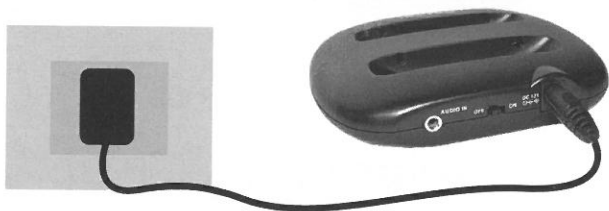


2. If your TV has audio output sockets connect the phono/phono lead to these sockets and plug the 3.5mm jack plug end of the lead into the EchoIR audio in socket (for sound boost connect to mic in socket). If your TV has no audio output sockets but has a spare SCART socket, plug the SCART adapter into the TV and connect the phono/phono lead to the SCART adapter then plug the 3.5mm end into the EchoIR audio in socket (for sound boost connect to mic in socket). If the SCART connection is not successful connect the microphone. Simply plug the 3.5mm plug of the microphone cable into the MIC IN socket on the EchoIR transmitter and then fix the microphone onto the TV speaker using the Velcro strip provided, please see picture below. It may also be possible to connect to a digital receiver such as a Sky or Virgin box, please see page 14 for further advice.

If your TV only has a Digital Optical Audio out socket, we recommend you purchase a Digital Audio Converter from our web shop www.hear4you.com.


Connect to audio out of TV

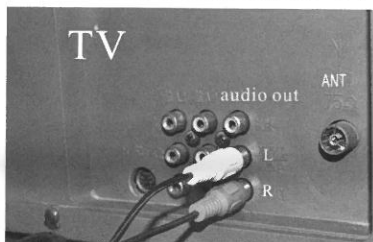


Microphone on lead attached to TV speaker with velcro® pad.

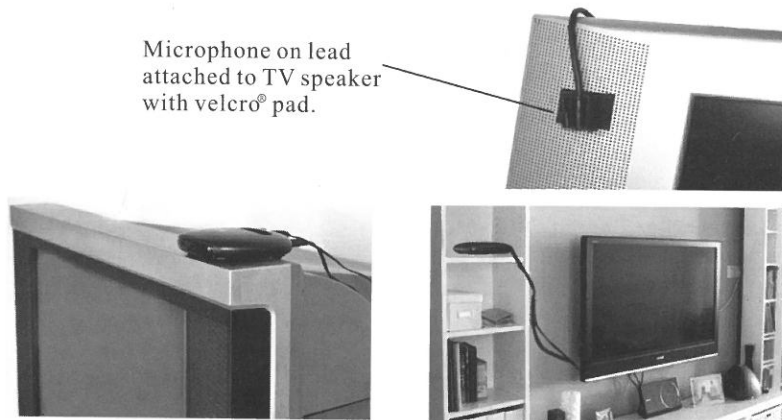

3. Ensure the transmitter is positioned so that it faces into the room and that there are no obstacles between the transmitter and the position the receiver will be used. Switch the transmitter on.

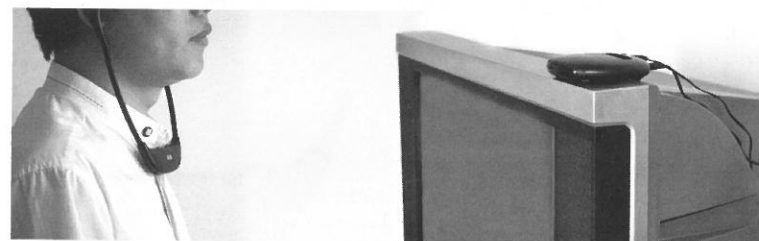

4. **IMPORTANT Make sure volume is set to off** (Volume wheel fully anticlockwise). Switch receiver on, the red power indicator light on the front of the receiver will come on. Wear the receiver with the infra red window facing the transmitter (power indicator facing the transmitter). Adjust the volume to a comfortable listening level by rotating the volume wheel clockwise.

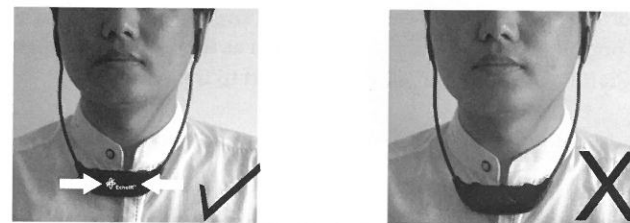

5. If using neckloop system, set hearing aids to the T setting, place neckloop over head and around neck then adjust volume to a comfortable listening level.